



Resin		(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Crown	Pro Cast Denture	5	13	8	ON
	Guide	6	10	8	ON
	Tray	7	13	8	ON
	Splint	10	6	8	ON
Model Temp	Pro Denture IBT	10	13	8	ON
	Pro Splint	15	8	1	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Element Beige Element White	20	2	5	OFF
Gingiva	20	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Guide	5	16	10	ON
Tray	7	16	10	ON
Flex Model	10	16	10	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Teeth Temp Crown plus	2min 30s	13	8	ON

* Flip the object and restart the post curing cycle



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Crown&Bridge Denture base Denture teeth Try-in	5	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Lumina Zero Denture Cast NF	5	13	8	ON
Guide	6	10	8	ON
Tray	7	13	8	ON
Splint	10	6	8	ON
IBT Temp Match Model Grey Mock up Model GB Model Ivory Ortho Model Laviva light-pink	10	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Clara	15	8	1	ON
Gingiva	20	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Tray	7	13	8	ON
Gingiva Model Beige Model Grey Model Transparent	10	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Ocean Blue	5	13	8	ON
Beluga White Orca Black Dolphin Grey	10	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
KeyGuide Key Try In	5	16	10	ON
KeyModel KeyTray KeyModelUltra KeyOrtho IBT KeyOrthoModel KeySplintSoft	10	16	10	ON
KeySplintSoft Clear	12	8	10	ON
KeySplintHard	15	10	5	ON
KeyMask	20	16	10	ON



Resin		(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast	Denture	5	13	8	ON
	Guide	6	10	8	ON
	Tray	7	13	8	ON
	Splint	10	6	8	ON
Model GB	Temp	10	13	8	ON
Ortho Model	IBT				
	Gingiva	20	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
CMFH	5	13	8	ON
Tray	7	13	8	ON
SG Model 2.0	10	13	8	ON
Ortho Flex	10	16	10	ON

NYTE 3D

Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast Press Model Sand	10	13	8	ON

+ SAREMCO

Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
CROWNTEC	6	16	10	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Splint Try in Bite Tray IBT SG Model	10	13	8	ON
Sculpture	20	9	2	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Cast&Press	5	13	8	ON
Tray	7	13	8	ON
Gingiva Model Plus UV Sand Model UV Smoky white Model Plus UV Grey	10	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
Splint comfort Model Beige Model Fast Splint Try-In SG Model 2.0 Tray	5	13	8	ON
Dentbase Cast	7	13	8	ON



Resin	(T) Time (min)	(P) Intensity	(D) Duty Cycle	(B) BL
VeriModel Golden Brown VeriModel Grey VeriModel Ivory VeriModel White	5	16	10	ON
VeriTray	7	13	8	ON
VeriCast VeriSplint VeriGuide	10	16	10	ON